

A gloved hand in a blue nitrile glove is using a pipette to add a red liquid to a test strip. The pipette is labeled 'STIMULATED' and 'LOT 7W 03'. The test strip is labeled 'ABTASCADIZO' and 'aPTT'. The background is a solid teal color.

CLINICAL CHEMISTRY

Blood Dispensers

ABTASCADIZO™
mumont, TX 77704


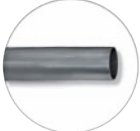







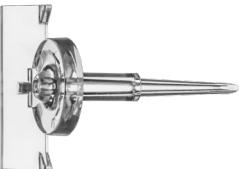








aPTT

800248
11/12

FOR MORE INFORMATION:
labcon.com/clinchem.html



Labcon offers several options for safe and efficient blood dispensing. Our Pette series reduce the risk of exposure to biohazards by dispensing blood without having to open the primary sample tube. Our Tip Top® caps offer a safer and easier alternative to pipetting from an open blood tube. Just carefully remove the stopper and replace with Tip Top®.

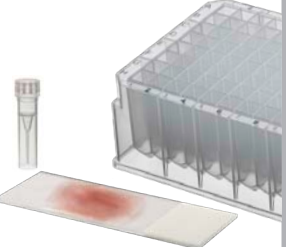Dispenser	Tip Style	Application(s)	Features	Fit
C-Pette® Low puncture force dispenser 	Stainless Steel  Blunt Tip	For Point-of-Care (POC) test cards & cartridges Works with most standard POC analyzers such as the Abbott® i-Stat®, & Helena® Abrazo®	Dispenses one precise drop of blood sample into test cards & analyzer cartridges safely & easily Steel cannula is designed for low puncture force	
C-Pette E® Economic blood dispenser 	Acrylic  Beveled Tip		Dispenses one precise drop of blood sample into test cards & analyzer cartridges safely & easily	
U-Pette™ Ergonomic blood dispenser 	Stainless Steel  Blunt Tip	 i-STAT is a registered trademark of the Abbott Group of Companies	Dispenses the required amount of blood sample: • one droplet at a time • or continuous flow Ergonomic, flexible sides provide better control of sample dispensing	
H-Pette® All-in-one dispenser with blade 	Acrylic  Beveled Tip	For blood smear preparation 	Dispenses one drop of blood sample onto slides safely & decreases steps to prepare thick or thin blood smears	
Tip Top® Filtered blood dispenser cap  13 mm  16 mm	Polyethylene  With filter	For blood smear preparation or dispensing into reagent wells or microcentrifuge tubes 	Dispenses one precise droplet or more of filtered blood through a small opening in the cap, providing a safe alternative to pipetting from an open blood tube	



C-Pette™ and C-Pette™ E Blood Dispensers

C-Pette™ and C-Pette™ E provide a quick and easy way to dispense blood into a test cassette without removing the cap from the collection tube. Both pettes fit all standard blood collection tubes and are ideal for use with many Point-of-Care (POC) analyzers that require the delivery of a sample to a test cartridge or cassette.

- Ideal for Point-of-Care (POC) use with any analyzer
- Minimizes aerosols and reduces the risk of lab-acquired infections
- Safe to use at patient bedside
- Fits all standard blood collection tubes



HOW TO USE THE C-PETTE™ AND C-PETTE™ E BLOOD DISPENSERS



GRASP AND INSERT



INVERT AND PLACE









PUSH AND DISPENSE

STEP 1: Grasp and insert the metal or acrylic cannula through the existing needle puncture of the capped blood collection tube.

STEP 2: Re-suspend cells by holding the tube by the plastic skirt and inverting gently as needed. Then place the C-Pette dropper tip into the cassette loading port.

STEP 3: Push down and release to fill cassette reservoir. If a larger sample volume is required, repeat push and release of tube.

Pette	Tip	Item #	Packaging	Qty/Pk	Qty/Cs
 C-Pette™	 Stainless Steel	7630-540-000-9	100 per box/1 box per case	100 pettes	100 pettes
	 Blunt Tip	7630-500-000-9	100 per box/5 boxes per case	100 pettes	500 pettes
 C-Pette™ E Includes inserter cover	 Acrylic	7625-540-000-9	100 per box/1 box per case	100 pettes	100 pettes
	 Beveled Tip	7625-500-000-9	100 per box/5 boxes per case	100 pettes	500 pettes



U-Pette™ Blood Dispenser

U-Pette provides a quick and easy way to dispense blood into a test cassette without removing the cap from the collection tube. This greatly reduces the risk of infection due to aerosol exposure. U-Pette's flexible sides allow for efficient and precise delivery of required sample amount, whether it's one droplet or continuous flow dispensing. Labcon's U-Pette fits all standard blood collection tubes and is ideal for use with many Point-Of-Care analyzers that require the delivery of a sample to a test cartridge or cassette.

- New ergonomic design for more control
- Ideal for Point-of-Care (POC) use with any analyzer
- Minimizes aerosols and reduces the risk of lab-acquired infections
- Safe to use at patient bedside
- Fits all standard blood collection tubes

HOW TO USE THE U-PETTE™ BLOOD DISPENSER



GRASP AND INSERT

STEP 1: Grasp the center of the U-Pette at the side of the disc as shown and insert the metal cannula through the existing needle puncture of the capped tube.



INVERT AND PLACE

STEP 2: Invert the tube, placing the tip of the U-Pette all the way into the sample well until it reaches the bottom.



PUSH AND DISPENSE

STEP 3: Use the tabs of the U-Pette to counter pressure as you push down on the tube to dispense sample as needed to reach the fill mark.

Pette	Tip	Item #	Packaging	Qty/Pk	Qty/Cs
U-Pette™ 	Stainless Steel	7626-540-000-9	100 per box/1 box per case	100 pettes	100 pettes
	Blunt Tip	7626-500-000-9	100 per box/5 boxes per case	100 pettes	500 pettes




H-Pette™ for Blood Smear Preparation

H-Pette™ dispenser makes blood slide preparation safe and easy. Since you don't have to open the primary sample tube, you greatly reduce the risk of lab-acquired infection by minimizing aerosols and exposure to blood borne pathogens.

H-Pette™ simplifies blood smear preparation by cutting out several steps, thereby improving efficiency, saving time, and reducing wasted materials and blood samples. Using the H-Pette eliminates the need for sticks, capillaries, or syringes to dispense blood on the slide. No need for special training, fittings, or inserts.

- Easy delivery of blood sample to a glass slide
- Minimizes aerosols and reduces the risk of lab-acquired infections
- No special training, special fittings or inserts
- Fits all standard blood collection tubes

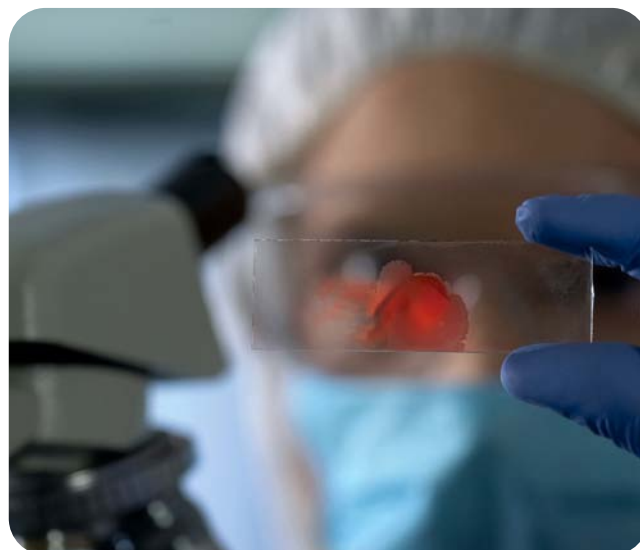

HOW TO USE THE H-PETTE™ ALL-IN-ONE BLOOD DISPENSERS



INSERT AND PRESS



DISPENSE ONE DROP ONTO GLASS SLIDE




SMEAR THE DROP ACROSS THE SLIDE

STEP 1: Insert H-Pette's cannula into the center of the stopper of the collection tube. Gently press until the cannula completely penetrates the septum. (The cannula is to be inserted in existing puncture in the stopper of the collection tube).

STEP 2: Invert collection tube with H-Pette's round edge pointing down, as shown. Gently press round edge to the glass slide to dispense one blood drop onto the slide.

STEP 3: Rotate tube at an angle with H-Pette's blade pointed down, as shown. Use the blade to make a smear just as you would a glass slide.

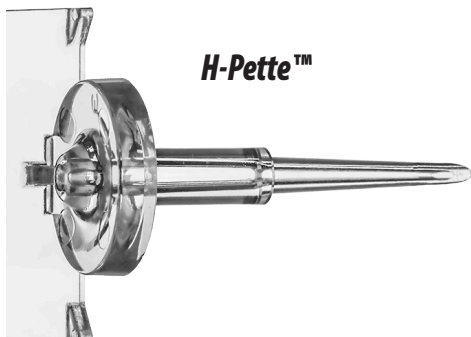Pette	Tip	Item #	Packaging	Qty/Pk	Qty/Cs
	Acrylic	7620-540-000-9	100 per box/1 box per case	100 pettes	100 pettes
	Beveled Tip	7620-500-000-9	100 per box/5 boxes per case	100 pettes	500 pettes



Tip Top® Blood Dispenser Caps


The Tip Top® is a disposable cone shaped cap that fits all standard 13mm or 16mm blood collection tubes. A small opening allows you to dispense a sample without removing the Tip Top® cap so you can prepare smears on slides or dispense into reagent wells or microcentrifuge tubes safely.


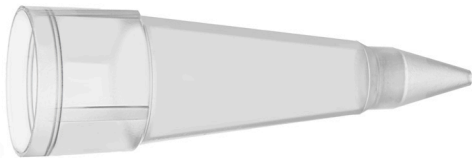
The Tip Top® cap can be used with almost any liquid. Made from chemically resistant polyethylene. They can dispense serum, reagents, plasma, whole blood, and Cerebrospinal fluid (CSF). The integral porous polyethylene filter efficiently separates serums and can also be used as a reagent blender.

- **Convenient** - just tilt the tube and squeeze to dispense one drop.
- **Safe** - no need to open the tube to pipette sample, reducing spillage along with potential exposure to infectious disease.
- **Low Evaporation** - a flexible cap is available to cover the Tip Top® so you can store samples in the refrigerator.



- Fits standard 13x100 mm and 16x100 mm collection tubes
- With integral filters for prep work

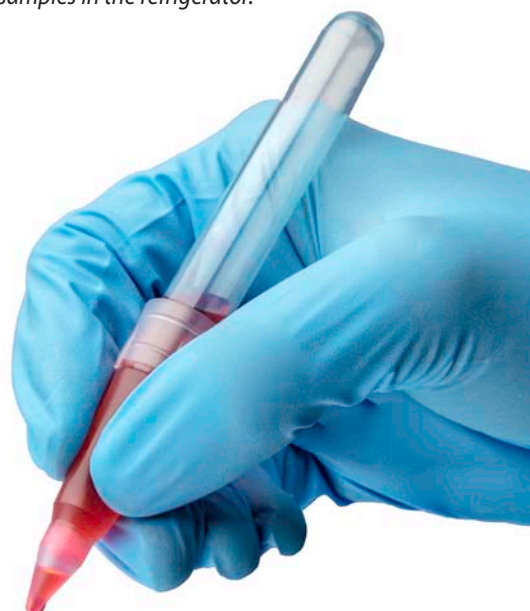

Dispenser	Tip	Item #	Packaging	Qty/Pk	Qty/Cs
Fits standard 13 mm collection tubes					
	Polyethylene	5511-800-000-9	1000 per bag/5 bags per case	1000 Caps	5000 Caps
Fits standard 16 mm collection tubes					
	With filter	5512-800-000-9	1000 per bag/5 bags per case	1000 Caps	5000 Caps